




PRECAUTIONS See-Green™ Lens Inserters and Stands

- ▲ Use the See-Green™ Lens inserter for Insertion only. Do not use to remove lens. (For removal, use the DMV Ultra Contact Lens Remover, available at <http://www.dryeyeshop.com>)
- ▲ Wash plunger and stand frequently in warm soapy water and allow to air dry. Do not wash stand or plunger in dishwasher.
- ▲ Wash hands thoroughly with soap (without moisturizer) and water, and dry before handling plunger and light.
- ▲ Before using, always wet the tip of the plunger with sterile saline. Do not apply a dry plunger to the lens surface.
- ▲ After use, clean tip of plunger, end of plunger and LED bulb with alcohol pad.
- ▲ CAUTION: Plunger contains natural rubber latex which may cause allergic reactions. Per manufacturer, DMV Corp., lab testing finds no antigenic proteins in this material. **CHOKING HAZARD : Keep this product away from small children.**
- ▲ **FOLLOW ALL LENS USE, CARE and INSERTION and Removal INSTRUCTIONS GIVEN TO YOU BY YOUR EYE DOCTOR!!!**

GENERAL CARE INSTRUCTIONS

- * The black battery supply case is designed to remain in the white box at all times. Keep the inside of the white box clean and dry at all times.
- * When inserting your lens, lay the wire over the slot in the end of the white box and close the cover over the wire as shown → 
- * Take the time to dry up your hands, the LED bulb and the wire before turning off the light and returning the wire to the white box. Leaving the light on for a few extra minutes during the process won't drain the battery
- * **DO NOT Leave the plunger on the LED (bulb) in between uses.** The plunger can trap saline against the LED and epoxy that protects the soldered connections to the LED. The saline is corrosive to both.
- * **DO NOT KEEP the inserter in a bathroom** with high moisture content ie, from showering. Doing so will shorten the life of your lens inserter. Store the unit in a dry place.
- * **DO NOT leave in a car in freezing or very hot temperatures.** Carry the unit on board air flights rather than in luggage in the cargo bay.

USE INSTRUCTIONS See-Green®™ Lens Inserter

To attach plunger to Light: Wet the bottom of the plunger with sterile saline. While holding the bottom of the plunger, insert the LED bulb in the small hole with a twisting motion so only the clear part of the LED bulb enters the plunger.




Inserting your lens: Follow your doctor's instructions for inserting your lens. Tips: **1)** The lid spread is everything! Hold the lids by the margins, where the lashes meet the lids. The contact points should be centered so as to form a perfect circle. Hold both lids open. Tuck your chin to your chest. Look straight down. Keep the plunger centered on your eye by focusing on the green light in the hole of the plunger. Continue approaching your eye smoothly and steadily. You will feel the fluid, but you are not done yet. Press lens onto the eye.




2) Capture the lens with the eyelids by releasing the lids. Don't squint. Maintain the upward pressure on the lens.
3) Release the lens from plunger by squeezing completely. Maintain the upward pressure against the eye, while squeezing to release the lens. The lens has now been released onto your eye. Stay squeezed while you retract the plunger away from the eye.

Push Upwards and Squeeze

Stay squeezed while you retract the plunger away

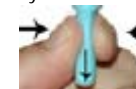

Plunger Stands

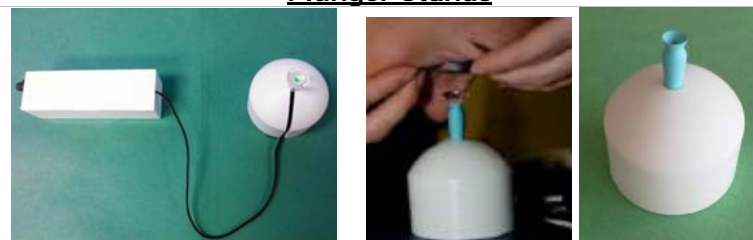

Our stands hold plungers and lenses securely while being prepared for insertion. They provide a base to steady your hands and allow you to get your eye close to the lens prior to lifting it up to the eye. The stand on the left is for use with the See-Green®™ Lens Inserter. The slot allows the wire to enter and to come up as the plunger is put on and lifted up from the stand. Some people need to use two hands to hold open their lids as shown in above. When using the stand this way, instead of raising the plunger to your eye, lower your head smoothly and steadily while focusing on the green light in the hole in the plunger until you will feel the fluid. Press your eye downward onto the lens. Capture the lens with the eyelids by releasing the lids. Lift up your head, and the lens should come off the plunger if there isn't too much suction. If necessary, release the lens from the plunger by squeezing the plunger as described in 3) above.



Innovative Scleral Lens Accessories
www.dalseyadaptives.net



U.S. Patent No. 7,163,245

See-Green®™
Lens Inserters
•
Stands



www.dalseyadaptives.net

Since 2007, we have helped thousands of people overcome the natural aversion and difficulties of inserting large scleral lenses. The See-Green®™ Lens Inserter and Stands have long-standing recognition by ophthalmic professionals as essential assistive tools that enable people to insert and wear scleral lenses. We provide them to the Boston Foundation for Sight and their PROSE Clinic Partners as well as Hospitals, University clinics and Optometry Colleges. Eye care practitioners all over the United States and in other countries consistently order our products.

Many people tell us that they can't get their lenses in without them. Customers report to us they can now insert their lenses independently and some no longer need home health workers or family members to help with insertion every day. Others who had given up on their scleral lens due to difficulties tried our products and tell us they re-opened the door to the scleral lens option, and we are told things like "it's a game changer" or "a life saver". A recent customer wrote: "This device is instrumental to my wellness and helping me have a normal day". A doctor in Miami said: "We pulled off a miracle with your product" as it enable his 90 year old patient with vision in one eye to insert his own lens.

The green LED light of the innovative See-Green®™ Lens Inserter will help you feel comfortable, help center your lens, and help prevent flinching and darting away of the eye. Our stands hold plungers and lenses securely while being prepared for insertion. People with unsteady hands or those that need to hold their eye lids open with both hands will find them helpful.

Strictly follow the instructions given to you by your doctor for handling and care of your lenses and for use of our products. Patients of the Boston Foundation for Sight and its PROSE Clinic Partners must follow all instructions at:
<http://www.bostonsight.org/PROSE-treatment/Prosthetic-Device-Use-and-Care>
Visit us at: <http://www.dalseyadaptives.net> for additional products and e-mail us at service@dalseyadaptives.com for service or replacements. Please carefully read and follow all of the use and care instructions. Feel free to contact us if you need help or have any questions.

Our mission is to continue producing, improving and delivering products that are easy to use, economical, and long lasting. We are committed to prompt customer service and support and help special needs customers with custom product adaptations tailored to their individual needs, such as extra bright lights for extreme low vision users. The See-Green™ Lens Inserter comes with a one-year replacement guarantee. It is our pleasure to make and sell these helpful products, and gratifying to know they make a big difference in the quality of life for so many people who use them every day.

John F. Dalsey, Manager, Dalsey Adaptives, LLC
1380 Main Street • Suite 202 • Springfield, MA 01103-1441

Tel. (413) 563-2477

info@dalseyadaptives.com